# Hemolyzer-5-Diff 5P

## Hemolyzer-5-Reagenz 5P

### Order information:

Catalog-No.	Content
HE5201	1 L
HE5202	5 L

### Intended use:

White blood cells-5-part reagent for automated analysis on Hemolyzer instruments.

Hemolyzer-5-Diff 5P is a stabilized and micro-filtered WBC-5-part reagent for quantitative determination of leukocytes (WBC), leukocyte five-part differentiation (LYM, MON, NEU, EOS, BAS) and hemoglobin (HGB) concentration measurement of human blood on Analyticon hematology analyzers.

Hemolyzer-5-Diff 5P should be used with Analyticon reagents only. Erroneous results may be obtained if the reagent is applied with third party reagents. Please refer to the instrument's User Manual for further information.

### Reagent Concentration:

Buffers	< 1,2 %
Sodium salts	< 3,4 %
Preservative	< 0,5 %
Stabilizers	< 0,5 %

in ion-free water

### Preparation and Stability:

Reagent is ready for use

Leave the reagent at room temperature for at least 12 hours before use.

#### Stability:

Unopened container:	Up to the expiration date	at 15 – 30 °C
Opened container:	120 days	at 15 – 30 °C

### **Do not freeze!**

### Reagent Placement:

Remove the inlet cap from the reagent container to be replaced. Connect the reagent inlet to the new reagent container.

Be sure that the color on each tube, reagent container label and connector in the back of the instrument match.

Avoid any dust or microbial contamination of the tubing and reagents.

Prime thoroughly the new reagent and measure backgrounds according to the instrument's User Manual.

When installing a new lot of reagent, recalibrate the instrument as specified in your User Manual.

### Specimen:

Hemolyzer-5-Diff 5P is intended for use with blood specimens collected by vein puncture in EDTA anticoagulant. Specimens for hematological analysis may be stored for up to 5 hours at 15-30 °C or up to 24 hours after collection when refrigerated (2-8 °C).

### Notes:

For in vitro diagnostic use only.

For trained laboratory professional only.

Exercise the normal precautions required for handling all laboratory reagents.

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