## Cleaner

**REF** 209-01-030-01 5 x 15ml Cleaner  
209-16-020-01  
209-09-030-00  
209-19-020-01  
CG0116  
81009013

**IVD**

### Intended Use

The Cleaner is intended for regular cleaning of the wash station, waste tube, pipettor and waste pump of coagulation analyzer.

### Content

5 x 15 ml Aqueous Solution

### Preparation

The Cleaner is ready to use. Before use make sure that there are no bubbles or foam inside the bottle.

### Don't shake !

For additional information concerning the use of the Cleaner please consult the operator manual of the coagulation analyzer.

### Storage and Stability

Storage temperature:

+15...30°C closed bottle

### Don't freeze !

### Protect from direct sunlight !

Expiry date:

The stated expiry date of the not opened Cleaner is shown on the label of the bottle.

Do not use the Cleaner after expiry date.

Durability after first use:

Durability of the Cleaner in a open bottle on analyzer	+20...30°C	7 days
Durability of the Cleaner if bottle is closed after use	+20...25°C	1 month

### Safety instructions:

P281 Use personal protection equipment.

P234 Keep only in original container

P501 Dispose of contents/container in accordance with local/regional/national/international regulation.

### First aid measures:

Inhalation: Take the affected person into the fresh air in an upright position to make breathing easier. Get medical attention if symptoms occur.

Contact with skin: In case of contact with skin wash with plenty of water. Get medical attention if symptoms occur.

Eye contact: Rinse open eyes several minutes with plenty of water. Get medical attention if symptoms occur.

Swallowing: Get medical attention if symptoms occur.

### Dispose:

Disposal of this product, solutions and any by-products should at all times comply with the requirements of environmental protection and waste disposal legislation and any regional local authority requirements.

Dispose of surplus and nonrecyclable products via a licensed waste disposal contractor. Waste should not be disposed of untreated to the sewer unless fully compliant with the requirements of all authorities with jurisdiction.

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**REF** 209-01-030-01 5 x 15ml Reinigungslösung  
209-16-020-01  
209-09-030-00  
209-19-020-01  
CG0116  
81009013

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