## Washing Solution

<b>REF</b>	209-01-060-01	CG0117	Washing Solution
	209-16-040-01	81009012	5 x 15ml
<b>IVD</b>	209-19-040-01		
	209-09-060-01		

### Intended Use

The Washing Solution is intended for regular cleaning of the pipettor of coagulation analyzers.

### Content

5 x 15 ml	Washing Solution	
	Buffered Salt Solution	
	Imidazole	0.1mol/l
	HCL	0.1mol/l
	Detergent	0,02%

### Preparation

The Washing Solution is ready to use. Before use make sure that there are no bubbles or foam inside the bottle.

### Don't shake !

For additional information concerning the use of the Washing solution please consult the operator manual of the coagulation analyzer.

### Storage and Stability

Storage temperature:  
+15...30°C closed bottle

Possible precipitation can be dissolved by warming up at 37°C for 5 minutes in a water bath.

### Don't freeze !

### Protect from direct sunlight !

Expiry date:

The stated expiry date of the not opened Washing Solution is shown on the label of the bottle.

Do not use the Washing Solution after expiry date.

Durability after first use:

Durability of the Washing Solution in a open bottle on analyzer	+20...30°C	14 days
---	------------	---------

### Hazard statements:

H360D May damage the unborn child.

### Safety instructions:

P281 Use personal protection equipment.

P234 Keep only in original container.

P308 + P313 If exposed or concerned: Get medical advice/attention.

P501 Dispose of contents/container in accordance with local/regional/national/international regulation.

### First aid measures:

Inhalation: Take the affected person into the fresh air in an upright position to make breathing easier. Get medical attention if symptoms occur.

Contact with skin: In case of contact with skin wash with plenty of water. Get medical attention if symptoms occur.

Eye contact: Rinse open eyes several minutes with plenty of water. Get medical attention if symptoms occur.

Swallowing: Get medical attention if symptoms occur.

### Dispose:

Disposal of this product, solutions and any by-products should at all times comply with the requirements of environmental protection and waste disposal legislation and any regional local authority requirements.

Dispose of surplus and non-recyclable products via a licensed waste disposal contractor. Waste should not be disposed of untreated to the sewer unless fully compliant with the requirements of all authorities with jurisdiction.

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