BioLyzer® Order information:

Catalog No.	BioLyzer	Contents		
B3831	200 / 600	R1	2 x	20 ml
		R2	2 x	6 ml
B3833	300 / 600*	R1	2 x	13 ml
		R2	2 x	4 ml

*only for instruments with a reagent tray for 20 / 70ml bottles

Intended use:

Immunoturbidimetric assay for the in vitro quantitative determination of human C4 in human serum and plasma.

Summary:

The complement system can be activated via the classical and an alternative route. Complement factor C4 participates in activation by the classical route. A decrease in C4 is common, but complete absence is rare. A lowered concentration or the complete absence of C4 occurs in immunocomplex diseases, systemic lupus erythematosus (SLE), autoimmune thyroiditis and juvenile dermatomyositis. The commencement of SLE in patients with C4-deficiencies can often be detected at a very early stage, and the course of the disease is milder than in patients with normal complement levels. Infections such as bacterial and viral meningitis, streptococcal and staphylococcal sepsis and pneumonia are associated with a decrease in C4.

Additional differentiation can be obtained by the determination of C4 when the level of complement factor C3 is low. If in such cases the protein concentration of C4 is normal, then an activation of the alternative route is likely. The main use of C4 determinations is in assessing the course of hypocomplement conditions. As an acute phase protein, C4 is produced to an increased extent during inflammatory processes. It is elevated in systemic infections, non-infectious chronic inflammatory conditions (primarily chronic polyarthritis) and physiological states (pregnancy). The elevation rarely exceeds twice the normal value and can mask a reduction in the current consumption.

A variety of methods, such as nephelometry, radial immunodiffusion and turbidimetry are available for the determination of complement factor C 4.

Test principle:

Immunoturbidimetric assay
Anti-C 4 antibodies react with antigen in the sample to form an antigen/antibody complexes which, following agglutination, are measured turbidimetrically.

Reagent concentration:

R1:
TRIS* buffer pH 7.8 100 mmol/l
Polyethylene glycol 4.5%
NaCl 150 mmol/l
Preservative

R2:
Anti-human C 4 antibody (rabbit); dependent on titer
TRIS* buffer pH 7.8 80 mmol/l
NaCl 150 mmol/l
Preservative
*TRIS= Tris (hydroxymethyl)-aminomethane

Preparation and stability:

R1: Ready for use.
R2: Ready for use.
Unopened kit components: Up to the expiration date at +2°C to +8°C
Onboard stability: R1 84 days
R2 84 days

Specimen:

Collect serum using standard sampling tubes
Na heparin-, Li-heparin plasma or EDTA-Plasma
Stability: 2 days at +20°C - +25°C
2 days at +4°C - +8°C
Centrifuge samples containing precipitate before performing the assay.

Notes:

For in vitro diagnostic use.
The material safety data sheet contains further safety-related information. It is available for download from our homepage <http://www.analyticon-diagnostics.com>.
Exercise the normal precautions required for handling all laboratory reagents.

Limitation interference:

Criterion: Recovery within ±10% of initial value.
Icterus: No significant interference up to an index I of 100 (approximate conjugated and unconjugated bilirubin: 100 mg/dl)
Hemolysis: No significant interference up to an index H of 1200 (approximate hemoglobin concentration: 1200 mg/dl).
Lipemia (Intralipid): No significant interference up to an index L of 400 (approximate triglycerides concentration: 800 mg/dl). There is poor correlation between turbidity and triglycerides concentration.
Rheumatoid factors up to 260 IU/ml do not interfere.
No high-dose hook-effect is seen up to a C 4 concentration of 500 mg/dl.
Monoclonal gammopathy sera of the IgA or IgM type can interfere with the C4 determination. The effect of drugs or their metabolic products on the test is not known in all cases. In case of doubt it is recommended to repeat the test after the medication has been discontinued. However, the medication may only be discontinued according to instructions of the attending physician.

Testing procedure:

Materials provided

- Working solutions as described above
- Additional materials required
- Calibrators and controls as indicated below
- 0.9% NaCl

Measuring/reportable range:

5 - 120 mg/dl (0.05 - 1.2 g/l)
Determine samples having higher concentrations via the rerun function using 0.9% NaCl solution as diluents.

Expected values:

10 - 40 mg/dl (0.1 - 0.4 g/l)*

* Reference values according to CRM 470 Protein Standardization

Each laboratory should investigate the transferability of the expected values to its own patients' population and if necessary determine its own reference range.
For diagnostic purposes the C 4 results should always be assayed in conjunction with the patient's medical history, clinical examinations and other findings.

Analytical sensitivity (lower detection limit):

3.7 mg/dl (0.037 g/l)
The lower detection limit represents the lowest measurable C 4 concentration that can be distinguished from zero.

Imprecision:

Reproducibility was determined using controls (n = 20). The following results were obtained:

Sample	Within run		
	Mean (mg/dl)	SD (mg/dl)	CV %
Sample 1	11.54	0.559	4.8
Sample 2	26.80	0.471	0.8
Sample 3	38.02	0.77	2.0

Sample	Run to run		
	Mean (mg/dl)	SD (mg/dl)	CV %
Human serum 1	43.15	0.605	1.4
Human serum 2	17.37	0.541	3.1
Human serum 3	33.39	0.548	1.6

Method comparison:

A comparison of the Analyticon Turbitex C4 (y) with a commercial obtainable assay (x) gave the following result.
 $y = 0.9603x + 0.4646$; $r = 0.9945$

Quality control:

Protein Control Level 1 3 x 1 ml #7661
Protein Control Level 2 3 x 1 ml #7662

The control intervals and limits must be adapted to the individual laboratory and country-specific requirements. Values obtained should fall within established limits. Each laboratory should establish corrective measures to be taken if values fall outside the limits.

Calibration:

Standardization: The C4 method was calibrated against the Reference Preparation CRM 470 (RPPHS - Reference Preparation for Proteins in Human serum).

Calibration Type: Logit-Log

S1: NaCl (0.9%)
S2-S6: Bio Cal® P Calibration Set 5 x 1 ml #1475

Calibration frequency

- Full recalibration is recommended
- after lot change
 - as required following quality control procedures

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*TRIS= Tris (hydroxymethyl)-aminomethan-hydrochlorid

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