### Order information:

Catalog No.	Contents			
1503	R1	1 x	65 ml	R2 20 x for 3 ml
151	R1	1 x	110 ml	R2 10 x for 10 ml

### Intended use:

In vitro test for the quantitative determination of Creatine Kinase (CK) in human serum and plasma.

### Summary:

Creatine kinase (CK) is a dimeric enzyme occurring in four different forms: a mitochondrial isoenzyme and the cytosolic isoenzymes CK-MM (muscle type), CK-BB (brain type) and CK-MB (myocardial type).




The determination of CK and CK-isoenzyme activities is utilized in the diagnosis and monitoring of myocardial infarction and myopathies such as the progressive Duchenne muscular dystrophy. Following injury to the myocardium, such as occurs with acute myocardial infarction. CK is released from the damaged myocardial cells. In early cases, a rise in the CK activity can be found just 4 hours after an infarction, the CK-activities reaches a maximum after 12-24 hours and then falls back to the normal range after 3-4 days. Myocardial damage is very likely when the total CK activity is above 190 U/l, the CK-MB activity is above 24 U/l (37°C) and the CK-MB activity fraction exceeds 6% of the total.

The assay method using creatine phosphate and ADP was first described by Oliver, modified by Rosalki and further improved for optimal test conditions by Szasz. CK is rapidly inactivated by oxidation of the sulfhydryl groups in the active centre. The enzyme can be reactivated by the addition of acetylcysteine (NAC). Interference by adenylate kinase is prevented by the addition of diadenosine pentaphosphate and AMP. Standardized methods for the determination for CK using the "reverse reaction" and activation by NAC were recommended by the German Society for Clinical Chemistry (DGKC) and the International Federation of Clinical Chemistry (IFCC) in 1977 and 1990 respectively. This assay meets the recommendations of the IFCC and DGKC (Standard method 94).

### Test principle:

UV Test

Sample and addition of R1:

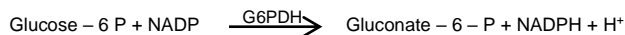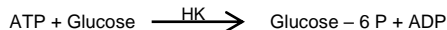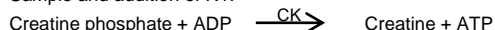Equimolar quantities of NADPH and creatine are formed at the same rate. The photometrically measured rate of formation of NADPH is proportional to the CK activity.

### Reagent Concentration:

<b>R1:</b>	
Imidazole buffer, pH 6.7	100 mmol/l
Glucose	20 mmol/l
Mg-Acetate	11 mmol/l
Brij 35P	< 3 %
Preservative/ Stabilizer	< 0,1 %
<b>R2:</b>	
ADP	5 mmol/l
AMP	5 mmol/l
Diadenosine P-5-P	10 µmol/l
NADP	2 mmol/l
Creatine phosphate	30 mmol/l
Hexokinase (HK)	2500 U/l
G-6-P-DH	1500 U/l
N-Acetylcysteine	20 mmol/l

### Preparation and stability:

Unopened bottles are stable until the end of the expiry when stored at 2–8°C.

Dissolve contents of enzyme reagent/R2 with the corresponding volume of buffer /R1. Gently swirl until completely dissolved. DO NOT SHAKE. Before first use wait approx. 10 min.

This working reagent is stable:

3 days	at +20°C to +25°C
15 days	at + 2°C to + 8°C

### Specimen:

Collect serum using standard sampling tubes.

Heparinized- or EDTA-plasma

Stability:	7 days	at + 4°C to + 8°C
	24 hours	at +20°C to +25°C

Centrifuge samples containing precipitate before performing the assay.

### Notes:

For in vitro diagnostic use.

Warning! R1 contains hazardous components.

The material safety data sheet contains further safety-related information. It is available for download from our homepage <http://www.analyticon-diagnostics.com>. Exercise the normal

precautions for handling all laboratory reagents.

### Limitation interference:

Criterion: Recovery within ± 10% of initial values.

Icterus: No significant interference up to an index I of 100 (approximate bilirubin concentration: 100 mg/dl). Hemolysis: No significant interference up to an index H of 1100 (approximate haemoglobin concentration: 1100 mg/dl).

Lipemia (Intralipid): No significant interference up to an index L of 850 (approximate triglycerides concentration: 1700 mg/dl). There is poor correlation between turbidity and triglycerides concentration.

Blood samples should only be drawn prior to the administration of sulfasalazine and sulfapyridine. Physiological plasma concentrations of sulfasalazine or sulfapyridine can lead to incorrect results.

The effect of drugs or their metabolic products on the test is not known in all cases. In case of doubt it is recommended to repeat the test after the medication has been discontinued. However, the medication may only be discontinued according to instructions of the attending physician.

### Testing procedure:

Materials provided

- Working solutions as described above

Additional materials required

- Calibrators and controls as indicated below
- 0.9% NaCl

### Manual procedure:

Wavelength:	340 nm (Hg 334 nm, Hg 365 nm)		
Temperature:	+37°C		
Cuvette:	1 cm		
Zero adjustment:	air or distilled water		

	Macro	Semi-Micro	Micro
Working reagent	2500 µl	1000 µl	500 µl

Incubate for 30 sec. at 37°C to pre-warm reagent. Then add:

Sample	100 µl	40 µl	20 µl
--------	--------	-------	-------

Mix and incubate for 3 minutes. Measure the absorbance increase per minute for another 3 minutes.

### Calculation:

	Macro	Semi-Micro	Micro
340 nm ΔE/min. x	4130	4130	4130
Hg 334 nm ΔE/min. x	4207	4207	4207
Hg 365 nm ΔE/min. x	7429	7429	7429

### Measuring /reportable range:

2 – 2300 U/l (0.03 – 38.41 µkat/l)

Manual procedure up to 1032 U/l (17.17 µkat/l). In case of higher activities, dilute sample 1:10 with 0.9% NaCl and repeat test. Multiply result by 10.

### Expected values:

Expected values on the basis of the optimized standard method of the DGKC (1977) and the IFCC (1990) method.

Myocardial infarction: There is high probability of myocardial damage when the following three conditions are fulfilled:

		25°C	30°C	37°C
1	CK <sub>Men</sub>	>80 U/l (1.33 µkat/l)	>130 U/l (2.17 µkat/l)	>190 U/l* (3.20 µkat/l)
	CK <sub>Women</sub>	>70 U/l (1.17 µkat/l)	>110 U/l (1.83 µkat/l)	>170 U/l* (2.85 µkat/l)
2	CK-MB	>10 U/l (0.17 µkat/l)	>15 U/l (0.25 µkat/l)	>25 U/l* (0.42 µkat/l)
3	The CK-MB-activity accounts for 6-25% of the total CK activity.			

\* Calculated values

If MI is suspected but the values obtained are below the specified limits, a fresh infarct may have occurred. In this case, tests should be repeated after 4 hours.

The following factors were used for converting the reference values from +25°C: 1.53 (+30°C) and 2.38 (+37°C).

CK varies with physical activity level and race in healthy individuals.

Each laboratory should investigate the transferability of the expected values to its own patient population and if necessary determine its own reference range. For diagnostic purposes the CK results should always be assayed in conjunction with the patient's medical history, clinical examinations and other findings.

### Analytical sensitivity (lower detection limit)

2 U/l (0.03 µkat/l)

The lower detection limit represents the lowest measurable CK concentration that can be distinguished from zero.

# CK NAc

## CREATINE-KINASE

### Imprecision:

Reproducibility was determined using controls. The following results were obtained:

Within run			
Sample	Mean (U/l)	SD (U/l)	CV %
Control serum 1	158	2.56	1.62
Control serum 2	462	3.37	0.73
Control serum 3	692	6.49	0.94

Between day			
Sample	Mean (U/l)	SD (U/l)	CV %
Control serum 1	145	6.64	4.55
Control serum 2	260	8.02	3.08
Control serum 3	424	9.83	2.32

### Method comparison:

A comparison of the Analyticon CK-NAc (y) with a commercial obtainable assay (x) gave the following result:

$$y = 0.977 x - 1.004; \quad r = 0.999$$

### Quality Control:

Human Control Serum

Contronorm® Plus	5 x 5 ml	#1205
	20 x 5 ml	#1220
Controptath® Plus	5 x 5 ml	#1305
	20 x 5 ml	#1320
Serum Control CK, CK-MB	2 x 2 ml	#1541

The control intervals and limits must be adapted to the individual laboratory and country-specific requirements. Values obtained should fall within established limits. Each laboratory should establish corrective measures to be taken if values fall outside the limits.

### Calibration:

S1: 0.9% NaCl		
S2: Bio Cal® E	10 x 3 ml	#1430
Bio Cal® Specific Enzymes	1 x 2 ml	#14575

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Text passages with grey background were changed in the latest revision of this package insert.

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