### Order information:

Catalog No.	Contents						
1903	R1	1 x	100 ml	R2	1 x	100 ml	
	R4	1 x	5 ml				
H3501	Hit I (ILab*)	R1	12 x	50 ml	R2	6 x	43 ml
H3503	Hit 917 (AU*)	R1	6 x	60 ml	R2	6 x	28 ml
AU3503	AU	R1	6 x	60 ml	R2	6 x	28 ml

(\* ) Kit contains only reagent barcode for Hitachi system.

### System information:

Hitachi 911: ACN 043  
Hitachi 917: ACN 749

For application on the Hitachi 912, please use an open system or contact the Analyticon Customer Support.

### Intended use:

In vitro test for the quantitative determination of calcium in human serum, plasma and urine.

### Summary:

The calcium content of an adult is somewhat over 1kg (25000mmol), i.e. about 2% of the body weight. Of this, 99% is present as calcium hydroxyl-apatite in bones and less than 1% is present in the extra-osseous ICS (intra-cellular space) or ECS (extracellular space). The calcium level in the ECS (approx. 100mmol) is in dynamic equilibrium with the rapidly exchangeable fraction of bone calcium. Calcium ions affect the contractility of the heart and the skeletal musculature and are essential for the function of the nervous system. In addition, calcium ions play an important role in blood clotting and bone mineralization. In plasma, calcium is bound to a considerable extent to proteins (approx. 40%), 10% is in the form of inorganic complexes and 50% is present as free (ionized) calcium. The body's calcium balance is regulated by the parathyroid hormone (PTH), calcitriol (CT) and calcitonin.


The test is used for the diagnosis and monitoring of hypocalcemia (calcium deficiency) and hypercalcemia (excess calcium) in serum. The characteristic symptom of hypocalcemia is latent or manifest tetany and osteomalacia. Hypocalcemia is due to the absence or impaired function of the parathyroid or impaired vitamin D-synthesis. Hypercalcemia is brought about by increased mobilization of calcium from the skeletal system (osteoporosis) or increased intestinal absorption. The majority of cases are due to primary hyperparathyroidism (pHPT) or bone metastasis of carcinoma of the breast, prostate or thyroid and bronchial carcinoma. The main significance of determining urinary calcium lies in the differentiation between hypercalciuria and hypocalciuria and the differential diagnosis of nephrolithiasis.

Complexometric methods are used in addition to atomic absorption spectrometry (AAS) for determining calcium. The following calcium determination is based on the reaction of calcium with o-cresolphthalein complexone in alkaline solution. Magnesium is masked with 8-hydroxy-quinoline.

### Test principle:

Colorimetric assay with endpoint determination and sample blank

- Sample and addition of R1 (buffer)
- Addition of R2 (chromogen) and start of reaction:

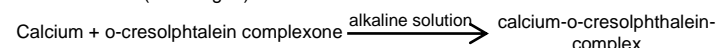The colour intensity of the purple complex formed is directly proportional to the calcium concentration and is measured photometrically.

### Reagent Concentration:

<b>R1:</b>	
AMP buffer, pH 10.7	350 mmol/l
<b>R2:</b>	
o-Cresolphthalein complexone	0.16 mmol/l
8-hydroxyquinoline	6.9 mmol/l
Hydrochloric acid detergent	0.06 mmol/l
<b>R4: (#1903)</b>	
Calcium	10 mg/dl (2.5 mmol/l)

### Preparation and stability:

#### Reagent-Start:

R1: ready for use  
R2: ready for use  
Unopened kit components up to the expiration at + 2 to + 25°C.  
Onboard stability: R1 35 days  
R2 35 days

#### Serum-Start:

Mix equal parts of R1 + R2 according to requirements.  
Stability of working solution:  
3 days at +15°C to +25°C  
7 days at +2°C to +8°C

### Specimen:

Collect serum using standard sampling tubes.  
Heparin plasma. Plasma must be assayed fresh.  
Stability: 7 days at +20°C to + 25°C  
3 weeks at + 4°C to + 8°C  
8 months at - 20°C

Do not use oxalate-, EDTA-, or citrate-plasma.

#### Urine

24-hour urine: Place 10 ml hydrochloric acid (6mol/l) in a collection bottle, or acidify (pH<2.0) after urine collection, in order to dissolve calcium salts. Before determination dilute urine 1:1 with distilled water, multiply the result by 2.

Stability : 2 days at +20°C to + 25°C  
4 days at + 4°C to + 8°C  
3 weeks at -20°C

Centrifuge samples containing precipitate before performing the assay.

### Notes:

For in vitro diagnostic use.

The material safety data sheet contains further safety-related information. It is available for download from our homepage <http://www.analyticon-diagnostics.com>. Exercise the normal precautions required for handling all laboratory reagents.

### Limitation interference:

Criterion: Recovery within ± 10% of initial value.

#### Serum/plasma

Icterus: No significant interference up to an index I of 100 (approximate bilirubin concentration: 100 mg/dl).

Hemolysis: No significant interference up to an index H of 200 (approximate hemoglobin concentration: 200 mg/dl).

Lipemia (Intralipid): No significant interference up to an index L of 100 (approximate triglycerides concentration: 200 mg/dl). There is poor correlation between turbidity and triglycerides concentration.

The effect of drugs or their metabolic products on the test is not known in all cases. In case of doubt it is recommended to repeat the test after the medication has been discontinued. However, the medication may only be discontinued according to instructions of the attending physician.

### Testing procedure:

Applications for automated systems are available on request.

#### Materials provided

- Working solutions as described above

#### Additional materials required

- Calibrators and controls as indicated below
- 0.9% NaCl
- Chimneys

<b>Manual Testing Procedure:</b>			
Wavelength:	Hg 578 nm (550-590nm)		
Reaction temperature:	+25°C		
Cuvette:	1 cm		
Zero adjustment:	One reagent blank for each series		
<b>Serum start:</b>			
	Blank	Calibrator	Sample
Working solution	2000 µl	2000 µl	2000 µl
Aqua dest.	50 µl	---	---
Calibrator	---	50 µl	---
Sample	---	---	50 µl
Mix and incubate 5 minutes. Then read the absorbance of sample and calibrator against reagent blank. Signal is stable for 5-60 minutes.			
<b>Reagent start:</b>			
	Blank	Calibrator	Sample
R1	1000 µl	1000 µl	1000 µl
Aqua dest.	50 µl	---	---
Calibrator	---	50 µl	---
Sample	---	---	50 µl
R2	1000 µl	1000 µl	1000 µl
Mix and incubate 5 minutes. Then read the absorbance of sample and calibrator against reagent blank. Signal is stable for 5-60 minutes.			
<b>Calculation:</b>			
ΔE Sample	x Conc. Calibrator = Calcium in mg/dl		
ΔE Calibrator			

### Measuring range:

#### Serum/plasma

Measuring range: 0.05-4.0 mmol/l (0.2-16 mg/dl)  
Determine samples having higher concentrations via rerun function. On instruments without rerun function, manually dilute samples with 0.9% NaCl or distilled/deionised water (e.g. 1+1). Multiply the result by the appropriate dilution factor (e.g. 2).

#### Urine

0.05-4.00 mmol/l (0.2-16 mg/dl)  
Roche/Hitachi 704/717  
Measuring range: 0.05-4.00 mmol/l (0.2-16 mg/dl)  
Roche/Hitachi 902/904/911/912  
Measuring range: 0.12-8.00 mmol/l (0.48-32 mg/dl)  
Roche/Hitachi 917

Measuring range: 0.12-9.25 mmol (0.48-37 mg/dl)  
Determine samples having higher concentrations via rerun function. On instruments without rerun function, manually dilute samples with 0.9% NaCl or distilled/deionised water (e.g. 1+1). Multiply the result by the appropriate dilution factor (e.g. 2).

### Reference value:

#### Serum/plasma

2.15-2.55 mmol/l (8.60-10.2 mg/dl)

#### 24-hour urine

2.5-8.0 mmol/24 h (100-320 mg/24 h), corresponding to 1.7-5.3 mmol/l\* (6.7-21 .3 mg/dl\*)

\*Assuming a urine volume of 1.5 l/24 h

### Reference range acc. to Tietz

#### Serum/plasma

Children (0-10 days): 7.6-10.4 mg/dl (1.90-2.60 mmol/l)  
Children (10 days-2 years): 9.0-11.0 mg/dl (2.25-2.75 mmol/l)  
Children (2-12 years): 8.8-10.8 mg/dl (2.20-2.70 mmol/l)  
Adults (12-60 years): 8.4-10.2 mg/dl (2.10-2.55 mmol/l)  
Adults (60-90 years): 8.8-10.2 mg/dl (2.20-2.55 mmol/l)  
Adults (> 90 years): 8.2- 9.6 mg/dl (2.05-2.40 mmol/l)

Urine: 100-300 mg/24 h (with normal food intake)

Each laboratory should investigate the transferability of the expected values to its own patient population and if necessary determine its own reference range. For diagnostic purposes the calcium results should always be assayed in conjunction with the patient's medical history, clinical examinations and other findings.

### Analytical sensitivity (lower detection limit):

Detection limit: 0.05 mmol/l (0.2 mg/dl)

The detection limit represents the lowest measurable calcium concentration that can be distinguished from zero.

### Imprecision:

#### Serum

Reproducibility was determined using controls. The following results were obtained.

Sample	Between day		
	Mean mg/dl	SD mg/dl	CV %
Control serum 1	8.17	0.13	1.59
Control serum 2	14.88	0.32	2.15
Control serum 3	29.77	0.58	1.95

Sample	Within run		
	Mean mmol/l	SD mmol/l	CV %
Control serum 1	2.21	0.033	1.49
Control serum 2	2.66	0.035	1.32
Control serum 3	3.89	0.031	0.80

### Method comparison:

A comparison of the Analyticon Fluitest® CA CPC (y) with a commercial obtainable assay (x) gave the following result:

$$y = 1.113 x - 0.278; \quad r = 0.99$$

### Quality control:

#### Human Control Serum:

Contronorm® Plus 5 x 5 ml #1205

20 x 5 ml #1220

Controptath® Plus 5 x 5 ml #1305

20 x 5 ml #1320

#### Urine Control:

Urine Control Set 8 x 5 ml #1507

The control intervals and limits must be adapted to the individual laboratory and country-specific requirements. Values obtained should fall within established limits. Each laboratory should establish corrective measures to be taken if values fall outside the limits.

### Calibration:

Standardization: The calcium method was standardized by atomic absorption spectrometry.

S1: 0.9% NaCl

S2: Bio-Cal® 20 x 3 ml #1420

Bio-Cal® E 10 x 3 ml #1430

R4: calibrator provided in kit.

#### Roche/Hitachi:

Absorption of atmospheric CO<sub>2</sub> by the opened reagent bottle R1 leads to impaired calibration stability. This kit therefore requires the use of colour-coded chimneys which reduce the uptake of CO<sub>2</sub> by the reagents. The chimneys should be placed directly into the appropriate reagent(s): white for R1. The chimneys can be reused for reagent bottles within the same kit.

Chimneys are used on all systems except the Roche/Hitachi 736/737 und 747. Calibration stability on these instruments can be achieved by the use of suitable caps for bottle R1

### Calibration frequency:

Two-point calibration is recommended:

- Every 7 days if the reagent bottles are onboard the analyser for more than 7 days
- After reagent bottle change if the previous bottles were onboard the analyser more than 7 days.
- After reagent lot change
- As required following quality control procedures

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