

Rinse solution

for Coagulyzer® Auto Pro



Order information:

Catalog-No.	Content
CG0121	3x1250 ml
CG0122	9x1250 ml

Approx. 1.000 tests/container (depending on profile and instrument setup)

Intended use:

The Rinse solution is intended to be used as a rinsing solution on the Coagulyzer® Auto Pro.

Summary and Explanation:

The Rinse solution is necessary for flushing the pipette needle (probe) in the Coagulyzer® Auto Pro.

Product Description:

Rinse solution is ready to use for flushing the pipette needle (probe) in the Coagulyzer® Auto Pro.

Warning and Precautions:

For in vitro diagnostic use only.

The material safety data sheet contains further safety-related information. It is available for download from our homepage <http://www.analyticon-diagnostics.com>.

For used Rinse solution (contaminated with sample and reagent), please avoid contact with skin and eyes. Wear suitable protective clothing and gloves. Dispose components in compliance with local regulations for infectious material.

Storage and Stability:

Rinse solution is stable until the expiration date on the container label when stored at 18 – 25°C.

Safety warning:

Only original branded Rinse solution from instrument manufacturer have the certification for use! Second source Rinse solution may cause wrong results. In this case operator lose any warranty and CE Certificate.

Symbols:



= for In Vitro Diagnostic Use



= read package insert



= the product complies with European legislation




= lot number



= catalogue number



= expiry

 = store at
18°C – 25°C

