

# Hemolyzer-3-Control

(Hematology controls)



## Order information:

Catalog-No.	Article	Content
HE3502	Hemolyzer-3-Control Set (L, N, H)	2 x 3 x 3 ml (L-N-H)
HE3503	Hemolyzer-3-Control Normal	6 x 3 ml (N)

## Intended use:

Hemolyzer-3-Control Set is a hematology control designed to monitor values on automated and semi-automated impedance type hematology analyzers. It can also be used for manual methods. Please refer to the assay table for specific instrument models.

## Summary and principle:

It is an established laboratory practice to use a stable control to monitor the performance of diagnostic tests. This control is composed of stable materials that provide a means of monitoring the performance of hematology blood cell counters. It is sampled in the same manner as a patient specimen.

## Control composition:

Hemolyzer-3-Control Set is an *in vitro* diagnostic reagent composed of human erythrocytes, simulated leukocytes and mammalian platelets suspended in a plasma-like fluid with preservatives.

## Storage and Stability:

Control is ready for use

Store Hemolyzer-3-Control Set upright at 2 - 8° C (35 - 46° F) when not in use.

Unopened tubes are stable until the expiration date. Opened tubes are stable for 14 days, provided they are handled properly.

## Protect tubes from overheating and freezing.

## Indications of deterioration:

After mixing, product should be similar in appearance to fresh whole blood. In unmixed tubes, the supernatant may appear cloudy and reddish; this is normal and does not indicate deterioration. Other discoloration, very dark red supernatant or unacceptable results may indicate deterioration. **Do not use the product if deterioration is suspected.**

## Instructions for use:

- Remove tubes from the refrigerator and allow to warm to room temperature (15 - 30°C or 59 - 86°F) for 15 minutes before mixing.
- To mix, hold a tube horizontally between the palms of the hands.  
**Do not pre-mix on a mechanical mixer.**
  - Roll the tube back and forth for 20 - 30 seconds; occasionally invert the tube. Mix vigorously, but do not shake.
  - Continue to mix in this manner until the red cells are completely suspended. Tubes stored for a long time may require extra mixing.
  - Gently invert the tube 8 - 10 times immediately before sampling.
- Analyze the sample as instructed in the Quality Control section of the Operator's Manual for your instrument.
- After sampling:
  - If tube has been open for sampling, clean residual material from the cap and tube rim with a lint-free tissue. Replace the cap tightly.
  - Return the tubes to refrigerator within 30 minutes of use.
- For consistency and best precision data, use the three levels of the cell control in the following order: abnormal low, normal, and abnormal high.

## Expected Results:

Verify that the lot number on the tube matches the lot number on the table of assay values. Assay values are determined on well-maintained, properly calibrated instruments using the instrument manufacturer's recommended reagents. Reagent differences, maintenance, operating technique, and calibration may contribute to inter-laboratory variation.

## Performance Characteristics:

Assigned values are presented as a mean and range. The mean is derived from replicate testing on instruments operated and maintained according to the manufacturer's instructions. The range is an estimate of variation between laboratories and also takes into account inherent imprecision of the method and expected biological variability of the control material.

Assay values on a new lot of control should be confirmed before the new lot is put into routine use. Test the new lot when the instrument is in good working order and quality control results on the old lot are acceptable. The laboratory's recovered mean should be within the assay range.

For greater control sensitivity each laboratory should establish its own mean and acceptable range and periodically re-evaluate the mean. The laboratory range may include values outside of the assay range. The user may establish assay values not listed on the Assay Sheet, if the control is suitable for the method.

## Performance Limits:

The performance of this product is assured only if it is properly stored and used as described in this insert. Incomplete mixing of a tube prior to use invalidates both the sample withdrawn and any remaining material in the tube.

## Notes:

For *in vitro* diagnostic use only.  
For trained laboratory professional only.

## WARNING - POTENTIAL BIOHAZARDOUS MATERIAL

The product contains human-sources and/or potentially infectious components. Each human donor/unit used in the preparation of this product has been tested by a FDA licensed method/test and found to be negative or non-reactive for the presence of HBsAg, Anti-HCV, NAT testing for HIV-1, HCV (RNA) and HIV-1/2. Each unit is also negative by a serological test for Syphilis (RPR or STS). Because no test method can offer complete assurance that infectious agents are absent, this material should be handled as potentially infectious. When handling or disposing of vials follow precautions for patient specimens as specified in the OSHA Bloodborne Pathogen Rule (29 CFR Part 1910, 1030) or other equivalent biosafety procedures

Exercise the normal precautions required for handling all laboratory reagents.

## Disposal:

Please note the legal regulations.