



Urilyzer 100 Pro

Software update:

- Test strip LOT and expiry management - The test strip LOT and expiry can be entered into the software and will appear on the measurement start button.
- Patient ID management - If desired, the instrument can be set to force the entry of the patient ID for each test.
- Settings management - Settings can be downloaded to an USB drive from one instrument and uploaded to another instrument.
- QC reference range input via Barcode reader - The LOT specific package insert for the CombiScreen Control PN includes a barcode. Reference ranges can be read and input easily into the software by reading the barcode.
- Operator management - The operator can be deleted, exported and imported.

More information here: [Urilyzer 100 Pro](#)